



## A New Approach for Management of the Shoulder

YOKE™ is a completely new approach to reduce pressure on supporting anatomical structures when using orthotic management to unload or stabilize the shoulder. Support straps are attached to a plastic yoke that is anatomically shaped to fit the contours of the trapezius and upper scapula. Pressure is evenly dispersed over the entire shoulder complex, significantly reducing pressure in the area of the acromioclavicular joint and brachial plexus.

- Reduce pressure on the clavicle, trapezius, and scapula.
- Versatile to meet treatment protocol - cuffs may be individually removed or added to meet individual patient exercise and strengthening protocols.
- Reduced Inventory - Fits left and right.



yoke™



### Prevent Shoulder Drop

A wide humeral cuff can be applied to the unaffected humerus as a counter pull to prevent shoulder drop when the paraplegic arm becomes too heavy.

### Elbow Stabilization

An elbow cuff may be applied to the affected side to stabilize the elbow and/or prevent the arm from sliding out of the sling.



### Comfortable Forearm and Wrist Support

Wide forearm and wrist support pads provide extended base of support for even pressure distribution and comfort. Horizontal connecting strap maintains parallel position for good alignment.

### Indications

- Subluxation due to stroke or other neurological disorders
- Shoulder pain
- Rotator cuff injuries
- Fractures in Humerus or Clavicle
- Subluxations due to injuries in the arm, Clavicle etc.
- For resting and off-loading after arthroplastic olecranon surgery

NOTE! Long time use can lead to contractions in the elbow.

**Product code 20050** One size

Washing instructions:  
All textile components can be removed and machine washed at 40°C (104°F).  
Close all Velcro prior to laundry. The plastic part can be cleaned with a wet towel.



**SVERIGE**  
Tel 042-25 27 01  
Fax 042-25 27 25  
mail.sweden@campscandinavia.se

**DANMARK**  
43 96 66 99  
43 43 22 66  
mail.denmark@camp.dk

**FINLAND**  
09-350 76 30  
09-350 76 338  
mail.finland@camp.fi

**NORGE**  
23 23 31 20  
23 23 31 21  
mail.norway@camp.no

[www.campscandinavia.se](http://www.campscandinavia.se)

