

TRIAL SLEEVE

General information about Elements Body can be found on our website. The trial sleeve is intended to be used as an evaluation tool to establish if the patient can be helped by an Elements Body product or not. The trial sleeve can be used up to two weeks during evaluation, maximum 8 hours per day. The prescriber's recommendation should be followed.

According to the table, the sleeve can be used by maximum 10 patients in evaluation purpose.

	Wrist	Elbow	Bicep
XXXS	11-12 cm	16-18 cm	16-18 cm
XXS	12-13 cm	18-20 cm	18-20 cm
XS	13-14 cm	20-22 cm	20-22,5 cm
S	14-15 cm	21-23 cm	22,5-25 cm
M	14-16 cm	22-25 cm	24-27 cm
L	16-18 cm	25-28 cm	27-30 cm
XL	18-20 cm	28-31 cm	30-33 cm
XXL	20-22 cm	31-34 cm	33-37 cm

When in doubt of size, rely on wrist measurement.

Refers to product with Serial Number: _____

NB! Ensure that the product is washed according to washing instructions between users.

Trial case	Delivery date	Patient ID	Return date	Washed and inspected (date & signature)
#1				
#2				
#3				
#4				
#5				
#6				
#7				
#8				
#9				
#10				

Inspection

- Product is intact and free from damages.
- Seams are intact.
- Washing label is still readable.
- If the test patient is an infectious patient, discard the trial sleeve.

Product description

Textile orthosis, sleeve in elastic material, no reinforcements.

Intended use and Indications

The Trial Sleeve is intended to be used for assessment purposes only, prior to the provision of Elements Body garment. The Trial Sleeve is intended for short term use for a maximum of two weeks and is a useful assessment tool within the clinic setting..

Contraindications

The Trial Sleeve is intended to be used for assessment purposes only, prior the provision of an Elements Body garment.

Limitations

- Usage no longer than two weeks.
- The product can be used by up to 10 patients for evaluation, two-week usage per patient.
- The fitter is responsible for providing a clean and damage free product to the next user.
- Use the product chart for documentation.

Cautions

- Should not be used damp.
- Check garment and skin daily before usage.
- Product should be washed and examined between change of users.
- Should be washed twice a week for maintained function.

Material Specifications

Polyamide, Elastane and Polyester
Anti-slip material in Thermoplastic Elastomer (latex-/silicone-free).

The fabric is certified according to standard 100 by Oeko-tex, class 2 and is GRS-certificated. Contains recycled fibres.

The product is not flame retardant treated.

Washing Instructions

Machine wash at 40°C.

Disposal

Disposal in accordance with local legislation.

Information

Any serious incident that may occur in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and / or patient is established.



Camp Scandinavia AB. Karbingatan 38
SE-254 67 Helsingborg, Sweden
Phone +46 42 25 27 01



allard

Sverige
042-25 27 01
info@camp.se
camp.se

Danmark
43 96 66 99
info@camp.dk
camp.dk

Suomi
09 350 76 30
info@camp.fi
camp.fi

Norge
23 23 31 20
info@camp.no
camp.no

UK/Ireland
42 932 8177
info@allarduk.co.uk
allarduk.co.uk

USA
888-678 6548
info@allardusa.com
allardusa.com

International
42 25 27 01
info@allardint.com
allardint.com