

39330/39333 I/39335/39336 Glove 39340 Sleeve Measuring Guidelines

elements
body



Orange & square = length

Blue & circle = circumference

All measurements should be taken in cm. Length measures, (orange square) – the measuring tape must be lying against the skin, but do not pull it. All measurements should be taken without tension, and directly against the skin. All measures should be taken when the Patient is relaxed and has a regular breathing pattern. It is useful to make marks using a non-permanent marker on the Patient's skin to secure correct measure


GLOVE

- L/R30 Circumference of Thumb at MCP Joint.
- L/R31 Circumference at end of required length of Thumb.
- L/R32 Circumference of Index Finger at PIP Joint (Long Fingered Glove) or where Finger should end on Glove (Short Fingered Glove).
- L/R33 Circumference at end of required length of Finger (Long Fingered Glove).
- L/R34 Circumference of Middle Finger at PIP Joint (Long Fingered Glove) or where Finger should end on Glove (Short Fingered Glove).
- L/R35 Circumference at end of required length of Second Finger (Long Fingered Glove).
- L/R36 Circumference of Ring Finger at PIP Joint (Long Fingered Glove) or where Finger should end on Glove (Short Finger Glove).
- L/R37 Circumference at end of required length of Ring Finger (Long Fingered Glove)
- L/R38 Circumference of Little Finger at PIP Joint (Long Fingered Glove) or where Finger should end on Glove (Short Fingered Glove).
- L/R39 Circumference at end of required length of Little Finger (Long Fingered Glove).
- L/R40 Circumference of Palm. Level with the Thumb web space
- L/R41 Circumference at MCP heads for Gloves with Fingers or top of Glove for Gloves without Fingers.
- L/R42 Length of Thumb from MCP joint to required length of thumb
- L/R43 Length from base of Index Finger to PIP Joint or to the required length of Index Finger (Short Fingered Glove). This measurement is taken on the ulnar border
- L/R44 Length from base of Index Finger to the required length of index Finger (Long Fingered Gloves). This measurement is taken on the ulnar border
- L/R45 Length from base of Middle Finger to PIP Joint or to the required length of Middle Finger (Short Fingered Glove). This measurement is taken on the ulnar border
- L/R46 Length from base of Middle Finger to the required length of Middle Finger (Long Fingered Gloves). This measurement is taken on the ulnar border
- L/R47 Length from base of Ring Finger to PIP Joint or to the required length of Ring Finger (Short Fingered Glove). This measurement is taken on the radial border
- L/R48 Length from base of Ring Finger to the required length of Ring Finger (Long Fingered Gloves). This measurement is taken on the radial border

Campscandinavia, DEC_2020©

39330/39333 I/39335/39336 Glove 39340 Sleeve Measuring Guidelines

elements
body

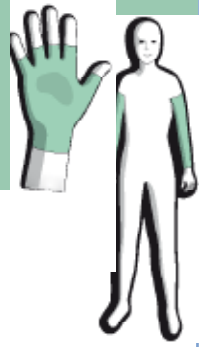

GLOVE

- L/R49 Length from base of Little Finger to PIP joint or to the required length of Little Finger (Short Fingered Glove). This measurement is taken on the radial border
- L/R50 Length from base of Little Finger to the required length of Little Finger (Long Fingered Gloves). This measurement is taken on the radial border
- L/R51 Length from base of thumb to base of Index Finger along the side of radial border of the Palm (Glove with Fingers) or to required length of Glove (Glove without Fingers)
- L/R52 Length from 2nd web space to Wrist crease over thenar eminence (Gloves with Fingers) or to required length of Glove (Gloves without Fingers).
- L/R53 Length from 4th web space to Wrist crease (Gloves with Fingers) or to required length of Glove (Gloves without Fingers).
- L/R55 Circumference at Wrist.

SLEEVE

- L/R54 Length from Wrist to Elbow joint (Long Sleeve) or to end of Sleeve (Short Sleeve). This measurement to be taken on the medial border of the Forearm
- L/R56A Circumference one third of the distance from Elbow to Wrist.
- L/R56B Circumference midway between Elbow to Wrist.
- L/R57 Circumference at Elbow joint with Arm extended (Long Sleeve) or at end of Sleeve (Short Sleeve).
- L/R58 Length from Axilla Crease to Cubital Crease. This measurement to be taken on the medial border of the upper Arm.
- L/R59 Circumference at midway point between Elbow and Axilla.
- L/R60 Circumference of Arm, level with Axilla. Patients Arms should be at their sides and Scapula set against Chest Wall.