


39300, 39301 Vest 39305, 39306 Body Vest / Body Sleeveless Measuring Guidelines

elements
body

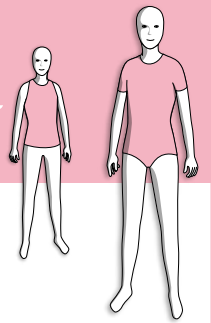

Orange & square = length

Blue & circle = circumference

All measurements should be taken in cm. Length measures, (orange square) – the measuring tape must be lying against the skin, but do not pull it. All measurements should be taken without tension, and directly against the skin. All measures should be taken when the patient is relaxed and has a regular breathing pattern. It is useful to make marks using a non-permanent marker on the Patient's skin to secure correct measure.

- U Umbilicus circumference
- C Chest circumference in line with Axillary Crease
- E Circumference of End of garment
- N Neck circumference around the base of Neck to lower edge of Sternal Notch.
- NS Neck Side length when wider Neckline is required. NS measurement cannot be greater than 1cm for children and 3cm for adults.
- ND Neck drop at front of Neck when a lower Neckline is required.
ND measurement cannot be greater than 3cm for children and 4cm for adults
- BD Depth at back of Neck when a lower Neckline is required.
BD measurement cannot be greater than 3cm for children and 4cm for adults.
- S This is the finished width required at Shoulder of Sleeveless garment. Remember if wider Neckline is required (NS) then this measurement must include NS.
- SU Lower edge of Sternal Notch to Umbilicus.
- AS Circumference level with Anterior Superior Iliac Spines (ASIS)
- UE Length from level with Umbilicus to where the garment should end. Take measurement along the side of body following the body shape. If Vest should end above Umbilicus record the length as a minus (-) measurement.
- WW Length from Waist to Waist through Legs. If Patient wears a nappy/pad, measure to side of nappy/pad.
- SCV Length from seventh Cervical Vertebra to Axillary Crease. To ensure an accurate measurement place a pencil or ruler under both Arms of Patient to ensure Arms remain against the body and the Scapulae are set against the Chest Wall.

Bust Measurements - Required for Women only. Take measurements with Patient standing / sitting wearing a bra.

- B Bust circumference at Nipple Line.
- UB Circumference under Bust.
- AUB Length from Axillary Crease to under Bust (UB)

Gastro Hole – Diameter of Gastro Holes are 6cm as standard. The measurements taken below indicate the position of the Gastro Hole. Complete for either Left or Right Side.

- GU Length from Umbilicus to Gastro Hole position above Umbilicus
- GD Length from Umbilicus to Gastro Hole position below Umbilicus
- GR Length from Umbilicus to Gastro Hole position on Right from Umbilicus
- GL Length from Umbilicus to Gastro Hole position on Left from Umbilicus.