
allard
Support for better life!





Instructions for Use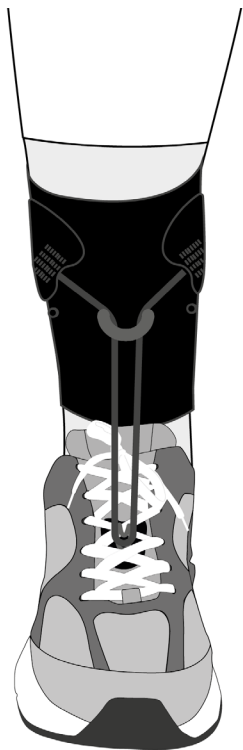

ToeUP™
Art. No. 28500

Instructions For Use

EN	ToeUP™ Textile ankle foot orthosis	p. 4
SV	ToeUP™ Textil ankel fot ortos	p. 6
NO	ToeUP™ Tekstil ankel fot ortose	p. 8
DA	ToeUP™ tekstil ankel fod ortose	p. 10
FI	ToeUP™ Tekstiilinen nilkka-ortoosi	p. 12
ES	ToeUP™ Ortesis textil de tobillo y pie	p. 14
DE	ToeUP™ Textile Knochel-/Fußorthese	p. 16
NL	ToeUP™ textiele enkel-/voetorthese	p. 18
FR	ToeUP™ Orthèse textile pour la cheville et le pied	p. 20
IT	ToeUP™ Supporto dinamico per piede equino	p. 22
ET	ToeUP™ Tekstiilist hüppeliigese ortoos	p. 24
LV	ToeUP™ Tekstila potītes pēdu ortoze	p. 26
LT	ToeUP™ Tekstilinė čiurnos pėdos ortožė	p. 28
JA	ToeUP™足関節装具(テキスタイル)	p. 30

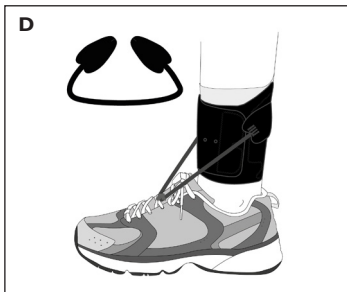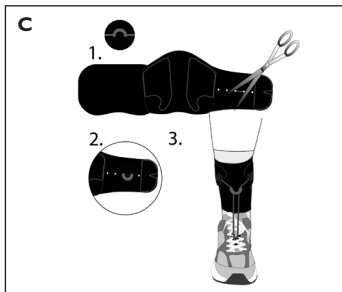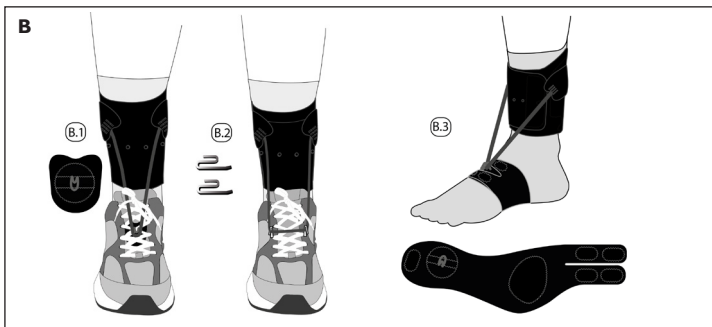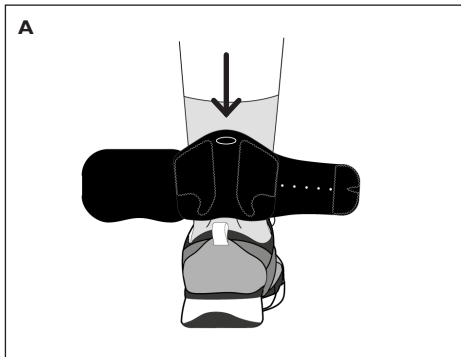EN ToeUP™ Textile ankle foot orthosis

Product description

ToeUP is a textile ankle foot orthosis designed for individuals with a mild form of foot drop. This orthosis features a low-profile textile ankle section that fastens above the malleoli (the bony prominence of the ankle). The ankle section is connected using an elastic cord with hooks, which are secured beneath the lacing or within the lacing holes on the shoe. This mechanism provides elevation for the forefoot during ambulation. You can adjust the level of elevation by altering the placement of the hooks or the strap on the ankle section. Additionally, there is a supplementary D-ring to fasten the straps together. ToeUP also offers an optional foot part component for indoor use. It is important to use ToeUP in conjunction with a sock.

Intended use

Designed for individuals with a mild foot drop. The extent of dorsiflexion assistance needed is easily adjusted using the incorporated elastic cord. Additionally, incorporating the ToeUP foot part allows for usage of the orthosis without footwear or shoes lacking an anchor point, such as sandals.
Single patient - multiple use.

Indications

For patients experiencing mild symptoms of foot drop due to neuro- and/or muscular weakness in dorsiflexion.

Contraindications

Should not be used by patients exhibiting open wounds, swelling, redness, or a significant buildup of heat, as well as severe oedema and persons sensitive to Latex.

Limitations

Do not prescribe to children under the age of three. The product must be fitted and/or adjusted by, or under the direction of, a healthcare professional.

Warnings

Avoid over-tightening the ankle section, as this could lead to reduced circulation and/or skin irritation. If you observe any alterations in skin colour or experience tingling sensations, discomfort, or chafing, remove the brace promptly. Ensure you examine your skin daily.

Prior to use

- ToeUP should be used together with a sock for best comfort and hygiene
- Check that the product is complete and without damage
- Ensure that the hook is clean from e.g., fuzz and hair
- Check the wear on the elastic cord before use

Material Specifications

Textile orthosis: Nylon, TPU, Spandex, PU, Polyester.

Details: Nylon, Polyester, Polyurethan, Stainless Steel, Steel coated with paint.

Elastic cord: Rubber, contains Latex.

This product is not flame retardant.

Washing Instructions

Machine wash at 40 degrees C. Close all hooks before wash. Wash separately in a washing bag. Lay flat to dry, loosen hook before drying. Do not tumble dry or dry clean.

Disposal

To be disposed in accordance with local legislations

Information

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

- A** Fasten the ankle section above the malleoli, place the ToeUP mark in the center back. If necessary, the edges at the top and bottom can be adjusted. The material can be cut with scissors by a healthcare professional.
- B** The elastic cord can be attached to the shoe tongue with hook under the lacing (B1), alternatively to the metal hooks in the lacing holes (B2) or to a foot part (B3) for indoor use. The ankle section is connected with the elastic cord in the hook that is attached to the foot part. The foot part is ordered separately for the right or left foot.
- C** For a more centered lift, place the hook with the D-ring in the center front. Cut up between the holes on the ankle part for best placement. This must be done in connection with fitting and by a healthcare professional.
- D** You can adjust the level of elevation by altering the placement of the strap on the ankle section or move the shoe tongue with hook forward or backwards under the lacing.

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Camp Scandinavia AB, Karbingatan 38