

## LCR - Lacer Wrist

|    |                                      |
|----|--------------------------------------|
| EN | <b>WRIST/HAND ORTHOSIS</b>           |
| DE | <b>HANDGELENK-/HANDORTHESE</b>       |
| FR | <b>ORTHÈSE DU POIGNET/DE LA MAIN</b> |
| ES | <b>ORTESIS DE MUÑECA/MANO</b>        |
| IT | <b>ORTESI PER POLSO/MANO</b>         |
| NL | <b>POLS-HANDORTHESE</b>              |
| DK | <b>HÅNDELED-/HÅNDORTOSE</b>          |
| FI | <b>RANNE-KÄMMENORTOOSI</b>           |
| SE | <b>HANDLEDS- OCH HANDORTOS</b>       |
| NO | <b>ORTOSE, HÅNLEDD/HÅND</b>          |
| PL | <b>ORTEZA NADGARSTKA/DŁONI</b>       |
| GR | <b>ΝΑΡΘΗΚΑΣ ΚΑΡΠΙΟΥ/ΧΕΡΡΙΟΥ</b>      |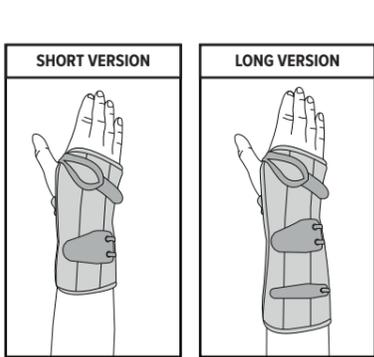

|  |  |
|--|--|
| <b>MD</b>  | <b>MEDICAL DEVICE</b>  |
| <b>1</b>   | <b>SINGLE PATIENT - MULTIPLE USE</b>   |
| <b>!</b>   | <b>CAUTION</b>   |
|  |  |
|  |  |
| Promedics Orthopaedics Ltd.<br>Port Glasgow Industrial Estate, Gareloch Road, Port Glasgow,<br>Scotland, UK, PA14 5XH<br>Tel: +44 (0) 1475 746 400   Email: enquiries@promedics.co.uk<br>www.promedics.co.uk |  |
| <b>CE</b>  | <b>EMERGO EUROPE</b><br>Westervoortsewijk 60,<br>6827 AT Arnhem<br>The Netherlands |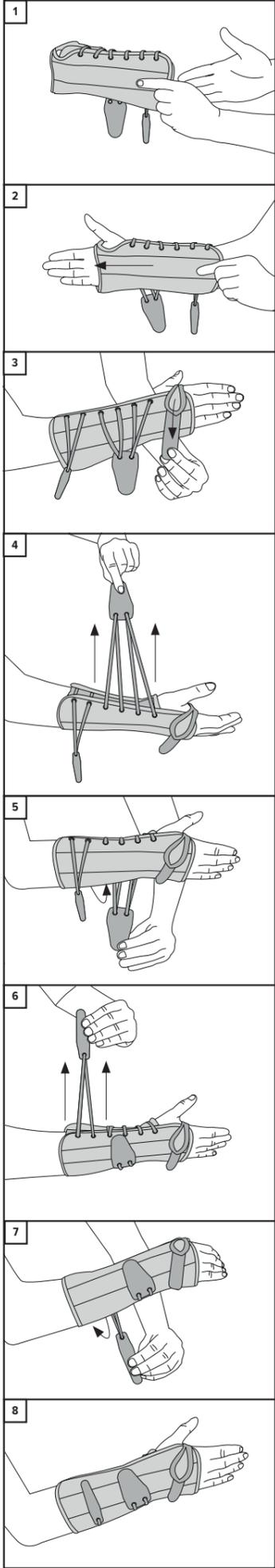

### ENGLISH

#### Product Description

An orthosis for support and stabilisation of the wrist. Conformable stay & wrist strap to aid stabilisation.

#### Usage Information (See diagram)

- Open the hook / loop straps and slip brace onto wrist.
- Position hand in brace as shown.
- Tighten laces as shown in diagram, wrap around wrist and close using hook fastener as shown in diagram. If product has an additional strap - repeat this process for the second hook / loop strap as shown in diagrams.
- Ensure end of brace is not above palmar crease.

#### Indications for Use

Provides wrist support for an array of injuries, conditions, pre-and post-operations such as:

- Arthritis.
- Sprains and strains.
- Soft tissue injuries and conditions.
- Instability.
- Post fracture management.

#### Contraindications

- Do not use over open wounds.
- Any known allergy and/or hypersensitivity to any material listed in product composition.
- Caution should be used if prescribed to patients with diabetes, vascular deficiency and neuropathy.

#### Warnings and Precautions

We recommend the initial fitting of this device be conducted by a suitably qualified healthcare professional who will advise the period of use.

- Carefully read all instructions and warnings prior to use.
- Follow all instructions to ensure proper performance of the brace.
- Do not use if liniments, ointments, gels, creams or any other substances have been applied to the affected area.
- Do not re-use for another patient, doing so risks cross infection and can compromise product integrity.
- Perform regular skin and circulation checks, especially patients with diabetes, vascular deficiencies and neurological conditions.
- Should any adverse reactions occur, please cease use and contact your healthcare professional or provider of this product.
- The durability of the device may be compromised by certain factors, e.g., objects with sharp edges or damage to the hook and loop fasteners. To prevent this the hook and loop fasteners should always be fastened when the device is not being worn or when being washed.

#### Washing and Care Instructions

- Close over all hook/loop straps & remove metal splint.
- Hand wash brace in cool water using mild detergent.
- Rinse and dry flat.
- Do not spin or tumble dry.
- Do not use fabric conditioner at any time.
- Check for any degradation that may affect use.
- Periodically check hook/loop fasteners are operating securely.
- Before using brace again, replace metal splint ensuring splint is correct way round and splint is tucked under edge / 'lip' of pocket.

#### Product Family Composition

Polyester, Polyurethane, Polyamide, Copper, Aluminium.

#### Storage and Transport Conditions

Store in a cool, dry place out of direct sunlight, in the original packaging.

#### Recycling and Disposal

Packaging and constituent parts should be recycled or safely disposed in accordance with local or national laws.

#### Serious Incident

Report any serious incident to the manufacturer and the competent authority of the EU Member state, or country in which you reside.

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