46029
VISSION WRIST
ORTHOSIS

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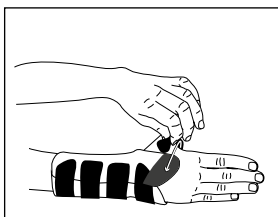
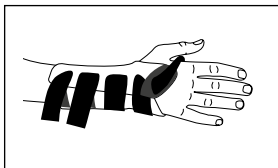
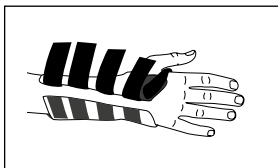
Indications / Intended Use

These products are intended for when the wrist needs support and off-loading in cases such as inflammations, work related strain or when rheumatic and/or degenerative changes are present.

Contraindications

No known

Adjustments

This product should only be altered by a medical professional person.

Warnings

Make sure that the straps are not too tight. If any sign of skin irritation occurs, immediately remove the support. Check your skin on a daily basis.

Material Specifications

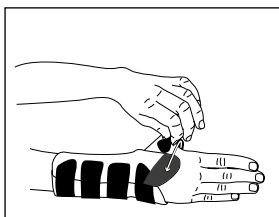
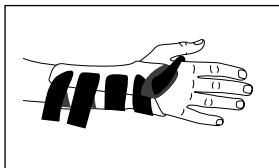
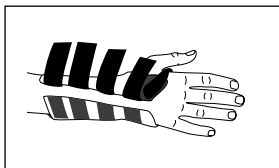
This orthosis is made from cotton, nylon, polyester, lycra and spandex. Spoon made in Polypropylene. This product is not flame retardant. Latex free.

Washing Instruction

Close all Velcro prior to laundering. Machine wash in 40°C (104°F), gentle cycle. Use a washing bag. Dry flat. Do not tumble dry or dry clean.

Who to Contact

If any problems occur, remove the support and contact your fitter or store where purchased.

**Instructions for use**

1. Check that the product is complete and without damage.
2. Open all Velcro straps.
3. Place the hand in the brace and make sure that the spoon in correct positioned in the palm of the hand.
4. Close the thumb strap.
5. Then close the other Velcro straps. Adjust the thumb strap.
6. Adjust all straps for comfort.

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