**46024 VISSION
THUMB ORTHOSIS
MOULDABLE**

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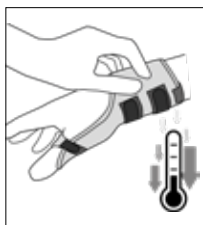
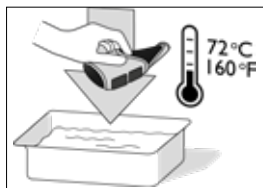
Carefully read this instruction sheet before use!

Intended use/Indications

This product is intended to restrict movements in CMC and MCP I joints and to position the thumb in a functional position by an individually moulded plastic thumb support.

Mild to Moderate CMC and MCP I instability, Ligament Injuries, Morbus de Quervain, Tendinitis, Degenerative diseases, Trauma. Intended for one persons use.

Contraindications

No known

Limitations

This product should only be altered by a medical professional person. The prescriber will determine the duration and conditions for use.

Warnings

Make sure that the straps are not too tight. If any sign of skin irritation occurs, immediately remove the support. Check your skin on a daily basis.

Material Specifications

Made from cotton, nylon, polyester, lycra and polyurethane. Low Temperature Thermoplastic from Copolymer. This product is not flame retardant. Latex free.

Washing Instruction

Remove plastic before wash. Close all Velcro prior to laundering. Machine wash in 40°C (104°F), gentle cycle. Use a washing bag. Dry flat. Do not tumble dry or dry clean.

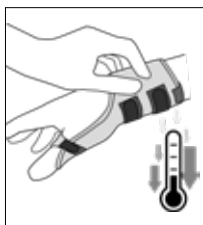
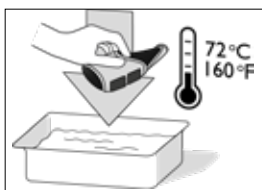
Who to Contact

Contact your fitter or store where purchased if you have questions regarding the use of the product.

Instructions for use

- 1. Check that the product is complete and without damage.**
2. Make sure the thermoplastic is correct positioned in the pocket.
3. Dip the complete product into a water-bath until the plastic is soft and mouldable.
4. Remove excess water with a towel and make sure the material is not too hot before fitting.
5. Shape the orthosis on the patient. Let the orthosis sit on the patient until cool. Remove and let it air-dry.
6. Adjust all Velcro straps on the patient.
- 7. During daily use the Velcro straps to open and close the orthosis. The plastic insert should be removed before washing.**

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