46023
VISSION THUMB
ORTHOSIS LONG
W/STAY

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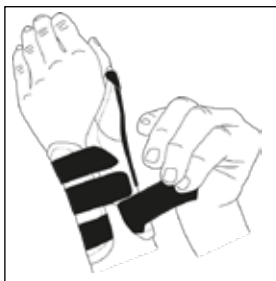
Carefully read this instruction sheet before use!

Intended Use/Indications

This product is intended to restrict movements in CMC and MCP I joints and to position the thumb in a functional position. Indications such as CMC and MCP I osteoarthritis or instability, Trigger Finger, Morbus De Quervain. Intended for one persons use.

Contraindications

No known

Limitations

This product should only be altered by a medical professional person. The prescriber will determine the duration and conditions for use.

Warnings

Make sure that the straps are not too tight. If any sign of skin irritation occurs, immediately remove the support. Check your skin on a daily basis.

Material Specifications

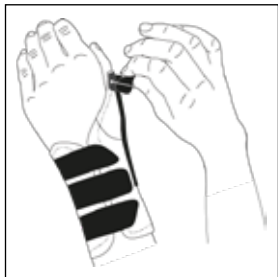
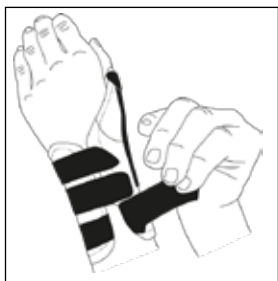
Made from cotton, nylon, polyester, lycra and polyurethane. Spoon made in Manganese. This product is not flame retardant. Latex free.

Washing Instruction

Close all Velcro prior to laundering. Machine wash in 40°C (104°F), gentle cycle. Use a washing bag. Dry flat. Do not tumble dry or dry clean.

Who to Contact

Contact your fitter or store where purchased if you have questions regarding the use of the product.

**Instructions for use**

1. Check that the support is complete and without damage.
2. Open the Velcro straps for the wrist.
3. Place the orthosis over the thumb.
4. Close the straps around the wrist. Then adjust the thumb strap.
5. Adjust all straps for comfort.
6. If needed, shape the stay on the thumb side.

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