
## allard





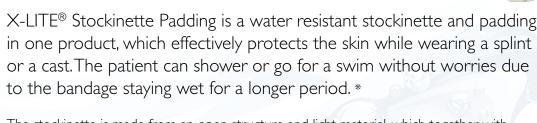



The stockinette is made from an open structure and light material, which together with X-LITE® plaster gives an extremely airy cast allowing the skin to breathe. The tubular knitted stockinette is made from polyester yarn with added Aerosilver fibers. The material effectively transports moisture away from the skin, ensures moisture evaporates

quickly and keeps the skin clean and dry. The Aerosilver fibres have an antibacterial function that reduces growth of odor causing germs, which usually occurs wearing a cast.

Apply the X-LITE® Stockinette Padding directly on the affected extremity in the right size avoiding wrinkles and folds. It is not necessary to use a regular stockinette underneath the Stockinette Padding. Always evaluate the patient and use a double layer for added protection. Apply the plaster as usual. The stockinette will keep its elasticity and pressure relief throughout the treatment.

Item No.	Colour	Width	Length	Unit
64444 0000	White	5 cm	10 m	1 Dispenser
64445 0000	White	7,5 cm 🕠	10 m	1 Dispenser
64446 0000	White	10 cm	10 m	1 Dispenser

\*Always consult attending physician before allowing the patient to swim with the cast and continuously evaluate the skin condition.



(