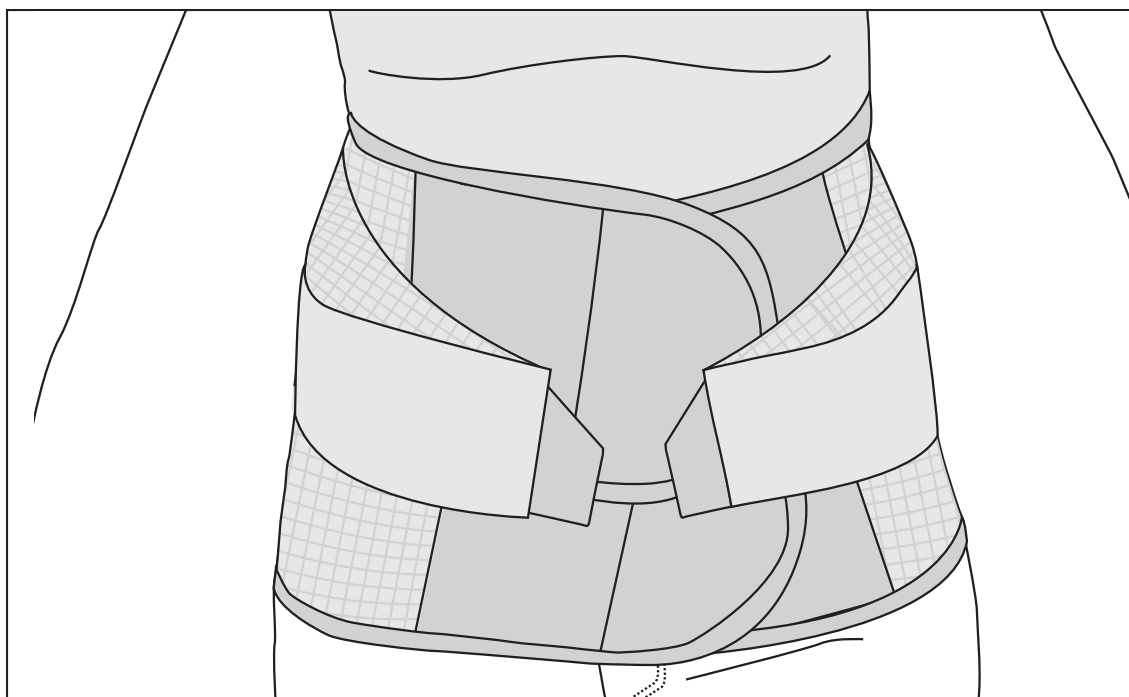
# allard

Support for better life!

## Instructions for Use

### Navy X™ Back High

Item No. 23552

NAVY X™

**Indications / Intended Use**

Elastic and airy back orthosis, made from extra firm and durable material. This model is extra high and features a split closure at the front for height adjustment. Strong tension straps provide extra stability, along with plastic stays in the sides and at the back. It has open pockets at the back for supplementary use of more stable stays, if required. Fastens with a Velcro closure at the center front. The orthosis is suitable for use in tough environments. Single patient - multiple use.

**Contraindications**

None found.

**Adjustment**

Fitting and modification of this product may only be performed by qualified healthcare professionals.

**Warning**

Ensure the skin is dry before use. Do not pull the straps too tight, as this may cause poor circulation or skin irritation. Check the skin daily and immediately remove the orthosis if you notice any discoloration in the skin, or if you feel any stinging, burning, or chafing. Contact the fitter if marks or chafing do not fade within a short time (30 minutes). Ensure these instructions are handed over to, and understood by, the user or the primary caregiver.

**Material specifications**

Polyester, Lycra, Nylon, POM plastic stays. This product does not contain Latex. The product is not flaming retardant.

**Wash instruction**

If the orthosis is equipped with extra stabilizing stays in the back, remove these and fasten all Velcro straps before washing. Use a laundry bag. Machine wash at 40°C, gentle cycle, using a mild detergent. Dry flat. Do not tumble dry, dry clean, iron or bleach.

**Disposal**

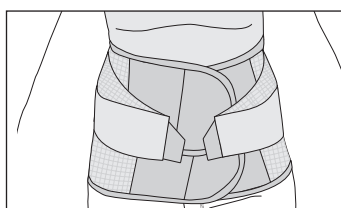
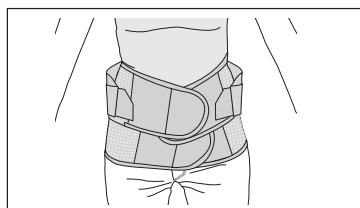
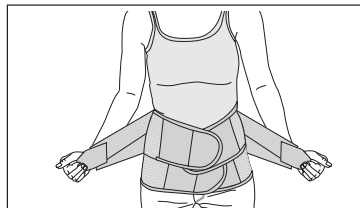
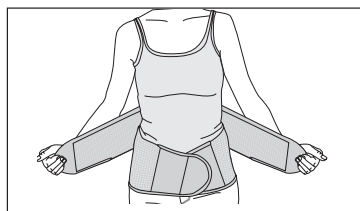
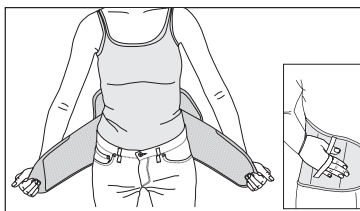
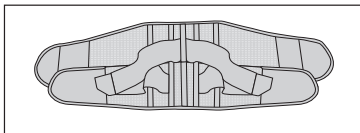
Disposal in accordance with local legislation.

**Information**

Any serious incident that may occur in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and / or patient is established.

**Who to contact**

If any problems occur please contact your orthotist or fitter.

**Instructions for use**

1. Check that the product is complete and without damage.
2. Start by placing the upper front closure on the small Velcro tabs located on the sides.
3. Put the orthosis on by closing the lower front closure. Make sure it is properly positioned and centered in the lower back area.
4. Close the upper front part on top of the lower one. Adjust the height and tension until it feels comfortable.
5. Detach the stabilizing tension straps and pull them forward for the desired stability. Fasten to the Velcro at the front.
6. To remove: First, slightly loosen the two stabilizing tension straps. Then open the upper Velcro closure and attach it on the small Velcro tabs located on the sides. Then open the closure.

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