allard

Support for better life!

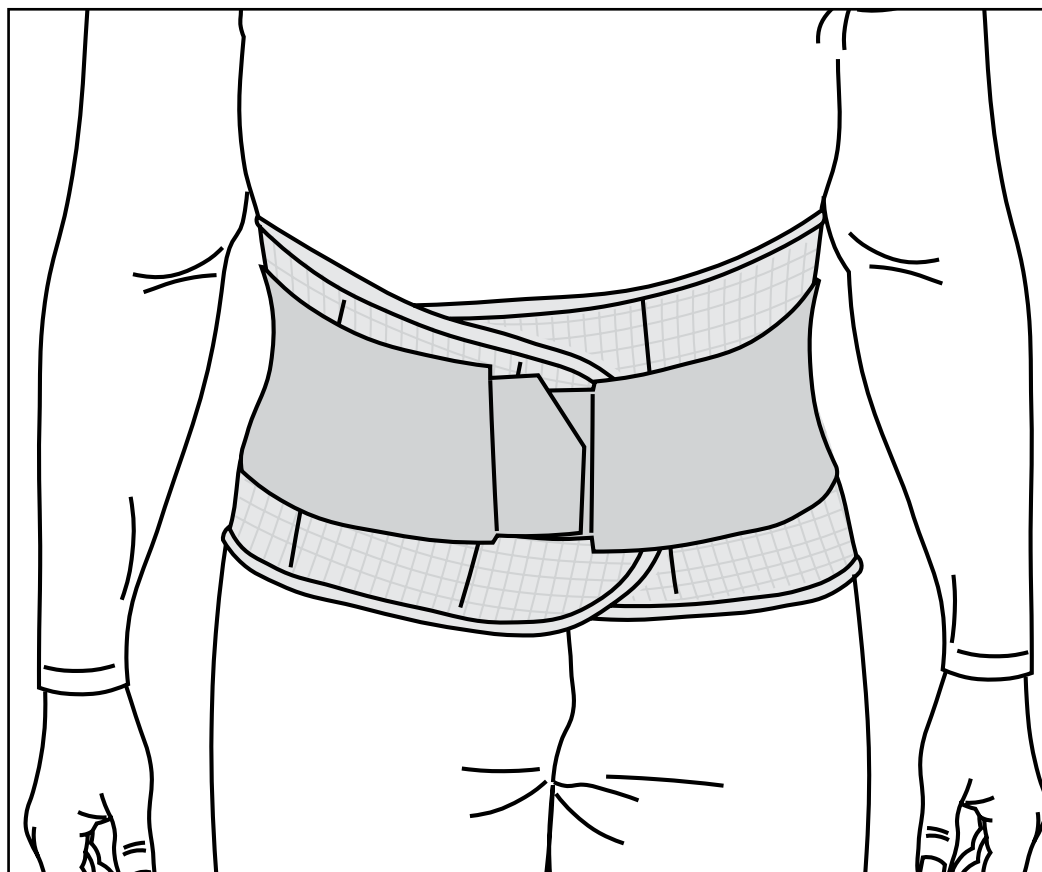
Instructions for Use

Navy X™ Back Low

Item No. 23549 (blue), 23554 (grey)

Navy X™ Back Regular

Item No. 23550 (blue), 23555 (grey), 23560 (beige)

NAVY X™

Indications / Intended Use

Navy X is intended for use with minor back problems, such as back pain and spinal insufficiency. It can also be used for preventive purposes during heavy lifting. Single patient - multiple use.

Contraindications

None found.

Adjustment

Fitting and modification of this product may only be performed by qualified healthcare professionals.

Warning

Ensure the skin is dry before use. Do not pull the straps too tight, as this may cause poor circulation or skin irritation. Check the skin daily and immediately remove the orthosis if you notice any discoloration in the skin, or if you feel any stinging, burning, or chafing. Contact the fitter if marks or chafing do not fade within a short time (30 minutes). Ensure these instructions are handed over to, and understood by, the user or the primary caregiver.

Material specifications

Polyester, Lycra, Nylon, POM plastic stays. This product does not contain Latex. The product is not flaming retardant.

Wash instruction

Fasten all Velcro straps before washing. Use a laundry bag. Machine wash at 40°C, gentle cycle, using a mild detergent. Dry flat. Do not tumble dry, dry clean, iron or bleach.

Disposal

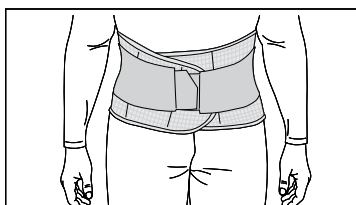
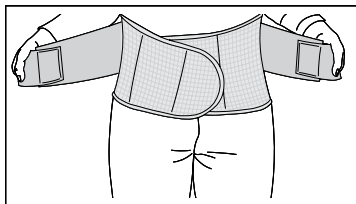
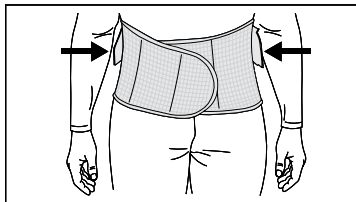
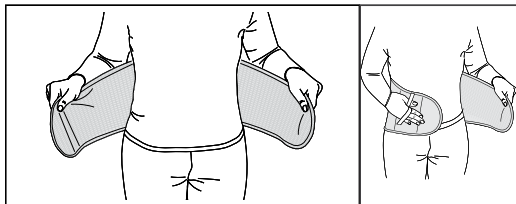
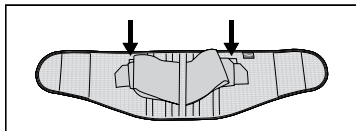
Disposal in accordance with local legislation.

Information

Any serious incident that may occur in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and / or patient is established.

Who to contact

If any problems occur please contact your orthotist or fitter.

**Instructions for use**

1. Check that the product is complete and without damage.
2. Start by placing the stabilizing tension straps on the small Velcro tabs located on the sides.
3. Put the orthosis on by closing the front closure. Make sure it is properly positioned and centered in the lower back area.
4. Adjust until it feels comfortable.
5. Detach the stabilizing tension straps and pull them forward for the desired stability. Fasten the straps to the Velcro at the front.
6. To remove: First, loosen the stabilizing tension straps and fasten them back onto the small Velcro tabs on the sides.
7. Then open the front closure.

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